## Specification

**Shelf Life:** Expiry date on product or individual packaging  
**FFP3 NR:** Single Use. Do not use for more than one shift  
**Protection duration:** ≤ 8 hrs  
**Size:** Standard

## Fitting Instructions

**IMPORTANT:** To provide the required level of protection, it is essential that the respirator is fitted correctly. Please follow the guidance below:

- Hold the respirator in your hand with the nosepiece at your fingertips, allowing the elasticated headbands to hang freely below your hand
- Press the respirator firmly against your face with the nosepiece positioned on the bridge of your nose
- Draw the headbands over your head one at a time. Position the top band high on the back of your head. Position the bottom band just below your ears
- Using both hands, mould the metal nosepiece to the shape of your nose
- To check the fit, place both hands completely over the respirator and inhale vigorously. If air leaks around the edges of the respirator, readjust the respirator, manipulating the nose bridge and headband to effect an acceptable seal. If a proper seal cannot be achieved DO NOT enter the contaminated area

Mark	Meaning
EN149:2001+A1:2009	Respiratory standard number
FFP1 NR	Class 1, low filtering efficiency - Concentrations up to 4 x OEL or 4 x WEL
FFP2 NR	Class 2, medium filtering efficiency - Concentrations up to 12 x OEL or 10 x WEL
FFP3 NR	Class 3, high filtering efficiency - Concentrations up to 50 x OEL or 20 x WEL

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## Note:

OEL = Occupational Exposure Limit  
 WEL = Workplace Exposure Limit

## Warning

The user should be trained in fitting the respirator and should be aware of the instructions for use prior to donning the respirator. Use in adequately ventilated areas, which are not deficient of oxygen and do not contain explosive atmospheres. The respirator must be fitted correctly to ensure the level of protection expected is achieved. It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.

Discard and replace the respirator if:

- The respirator is removed whilst in the contaminated area
- Clogging of the respirator causes breathing difficulties
- The respirator becomes damaged

FAILURE TO FOLLOW THE INSTRUCTIONS AND WARNINGS ON THE USE OF THIS RESPIRATOR DURING ALL TIMES OF EXPOSURE CAN REDUCE THE EFFECTIVENESS OF THE RESPIRATOR AND COULD RESULT IN ILLNESS OR DISABILITY.

## Storage and Shelf Life

This respirator should be stored in a container, protected from dust, chemicals, moisture, away from direct sunlight and in a clean, cool and dry atmosphere with humidity factor less than 75% and in a temperature between +2°C and +55°C. Failure to observe these requirements may affect shelf life and performance.

## Cleaning and Maintenance

Do not carry out any cleaning and maintenance procedures on this product. Do not use this product if it has not been stored correctly and/or is not in a fit state for use. Dispose of the product after use.

## Disposal

After use, carefully remove the mask in a clean, still environment and place in a sealable bag prior to disposal.

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Declares that

**Identification code:** 598550

**Description:** Respirator Fold Flat Valve

**Conforms to the following directives and standards:**

- PPE Directive 2006/42/EC
- EN 149:2001+A1:2009

**Notified body:** INSPEC International Ltd, Manchester, United Kingdom

**The technical documentation is kept by:** Silverline Tools

**Date:** 23/04/14

**Signed:**

Mr Darrell Morris

Managing Director

**Name and address of the manufacturer:**

Powerbox International Limited, Company No. 06897059. Registered address: Central House, Church Street, Yeovil, Somerset BA20 1HH, United Kingdom.

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**Keuringsinstantie:** INSPEC International Ltd, Manchester, United Kingdom